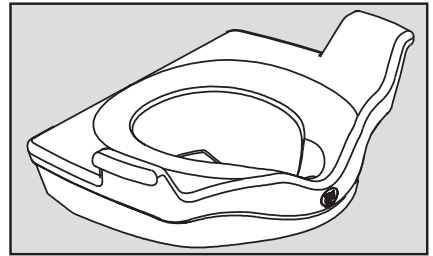
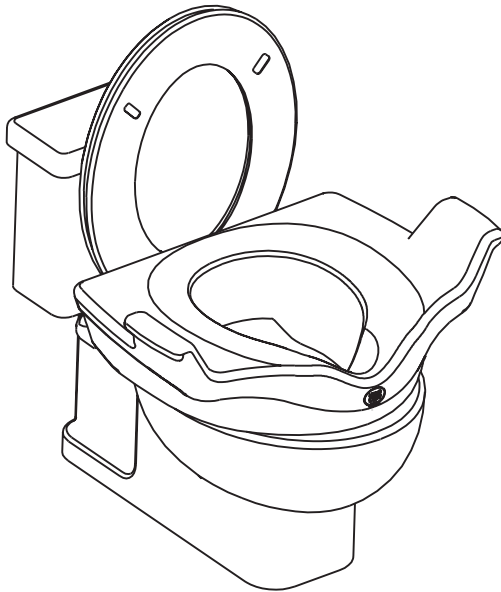
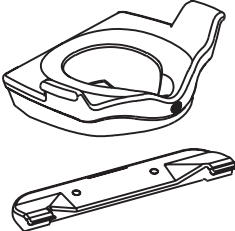
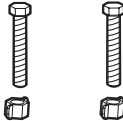
**LOCKING ELEVATED TOILET SEAT WITH SUPPORT HANDLES
INSTRUCTION SHEET**

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Warning ! – FOLLOW THESE INSTALLATION INSTRUCTIONS CAREFULLY. PROPER INSTALLATION IS THE INSTALLER'S RESPONSIBILITY. FAILURE TO FOLLOW CORRECT INSTALLATION PROCEDURES CAN CAUSE THE SEAT TO BECOME LOOSE, WHICH CAN RESULT IN SERIOUS INJURY.

Specifications:

Fits on most commode configurations – either round or elongated bowl. Raises seat 5 inches. Toilet seat must be clean and dry before installation. Plastic rim of seat must fit inside of toilet rim.

Dimensions:

24" wide x 16" deep x 5" high

Installation Instructions:

Tools Required: Standard size screwdriver to attach the bracket.

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1

Remove standard toilet seat and lid from toilet bowl.

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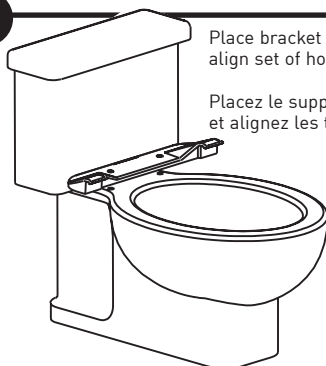
2

Place bracket on toilet bowl and align set of holes.

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3

Reattach toilet seat and lid with mounting bracket using supplied screws by lining up the holes on the toilet bowl.

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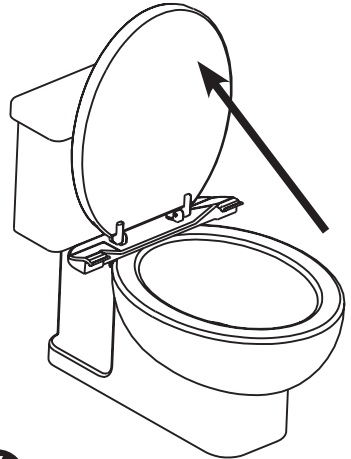
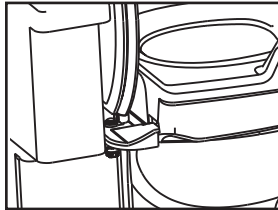
**5**

Attach the elevated toilet seat to the bracket on toilet bowl.

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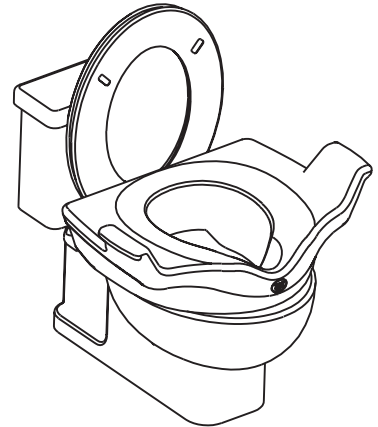
Push the seat towards the bracket to lock.

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To remove, reverse process, lift front of seat and pull.

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WARNING: TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 300 POUNDS. Consult healthcare professional for advice and instructions specific to your needs before use. Plastic inner rim of seat must fit inside of porcelain toilet bowl rim. If seat can rock from side to side, do not use. Do not tip seat. Sit down on and leave seat from front, not from side. When seated, do not lean to either side of seat without proper balancing. Do not jump or stand on seat.

WARNING: PREVENT INJURY. After elevated seat installation, both bracket tabs must securely engage slot in back of elevated seat, and there should be at least a small gap, no thinner than a dime, between the elevated seat and the bottom of the toilet's own seat. Do not use elevated seat without both bracket tabs securely engaged, or without gap.

CONTACT MOEN IMMEDIATELY IF ANY COMPONENTS ARE MISSING OR DAMAGED OR FOR ASSISTANCE WITH INSTALLATION.

LIMITED LIFETIME WARRANTY: If this product fails due to materials or workmanship at any time during the life of the product, [MOEN], Moen Incorporated, will replace it free of charge, postage-paid. Simply contact MOEN at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, MOEN DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND MOEN SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL MOEN'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.moen.com

Home Care by Moen Grab Bars meet or exceed the applicable requirements of the following standards/guidelines: The Americans with Disabilities Act (ADA)

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Home Care is registered trademarks of Moen Incorporated ©2016 • www.moen.com